PULPDENT®

Silane

Bond Enhancer


R_x only

Store in cool, dry place
MADE IN USA

CONTENTS:

- 4 1.2ml syringes silane applicator tips
- Instructions for use
- MSDS

To enhance bonding of composites and other organic resins to porcelain

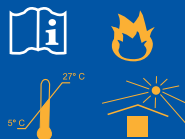 **PULPDENT Corporation** 80 Oakland Street • Watertown, MA 02472 • U.S.A
www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT®

Silane
Bond Enhancer

PULPDENT®

Silane Bond Enhancer

CE
0459

PULPDENT SILANE is a single-component material which is used to promote increased bond strength between organic resins and porcelain. To achieve desired results, the etched porcelain surface must be completely dry and free of residual moisture before applying silane.

CONTROLLED DISPENSING

Syringe dispensing with Pulpdent's unique red dropper tips eliminates waste and provides controlled dispensing of silane.

DIRECTIONS FOR USE

- Carefully follow instructions for Pulpdent Porcelain Etch Gel, and etch porcelain surface to be treated. Rinse and dry.
- Apply Pulpdent Dry-Rite™ drying agent to etched surface to assure that it is completely dry. Residual moisture will adversely effect bonding.
- Place 22-gauge red dropper tip on silane syringe and apply Pulpdent Silane Bond Enhancer to treated porcelain surface. Allow to dry. A gentle flow of uncontaminated, oil-free and moisture-free air may be applied to accelerate drying. Cap syringe immediately after use. To check air line for contamination, blow air onto the surface of dental mirror. If streaks of water or droplets of oil appear, do not use the air line for this procedure.
- Maintain a clean, dry field. Apply unfilled resin bonding agent and proceed with your preferred restorative composite or resin cement.

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