Porcelain Etch Gel

9.6% Hydrofluoric Acid

• Thixotropic

Rx only

Store in cool, dry place

MADE IN USA

CONTENTS:

4 1.2ml syringes etching gel

Applicator tips

Instructions for use

MSDS

 PULPDENT Corporation • 80 Oakland Street • Watertown, MA 02472 U.S.A.
www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT®

Porcelain Etch Gel

9.6% Hydrofluoric Acid *Does not stain resins or resin cements.*

PULPDENT PORCELAIN ETCH GEL is a 9.6% hydrofluoric acid gel for etching porcelain crowns, veneers and inlays or for etching fractured porcelain surfaces prior to cementation or bonding with resins and resin cements. It can also be used to etch hybrid composites.

DIRECTIONS FOR USE

1. Protective eyewear must be worn by practitioner, assistant and patient. Protective gloves must be worn by practitioner and assistant.
2. Place rubber dam to isolate operative site and protect soft tissue.
3. Assistant must stand ready at chairside and hold vacuum tip 2-3 inches from porcelain surface during the etching process to evacuate acid fumes.
4. Securely attach 25-gauge luer lock applicator tip to syringe. Carefully start the flow of Porcelain Etch Gel on a pad and immediately dispose of this sample. Do not start flow in patient's mouth. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between thumb and forefinger while resting plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Use a new dispenser needle for each patient.
5. Carefully apply Porcelain Etch Gel to porcelain surface and leave for one minute while evacuating fumes with vacuum. Some ceramic materials are resistant to etching. To achieve a surface suitable for bonding to these materials, air abrasion is recommended. Creating mechanical retention is advantageous. Cap syringe immediately after use.
6. Rinse surface with copious amounts of water and dry porcelain surface with clean uncontaminated air.
7. **If using Embrace™ First-Coat™**, follow instructions for use for First-Coat™ or the Embrace™ Restoration and PFM Repair Kit. **If using Pulpdent Silane Bond Enhancer**, follow instructions for use for silane.
8. Proceed with repairs of fractured porcelain teeth or with cementation of porcelain crowns, veneers or inlays.

Note: Multi-dose syringes should either be encased in a fresh protective barrier for each patient or cleaned and disinfected between patients, as appropriate

CAUTION: Porcelain Etch Gel contains 9.6% hydrofluoric acid, a powerful and aggressive chemical that produces irritating fumes and must be handled with extreme care. Ingestion, contact with skin and eyes, and chronic inhalation of fumes are dangerous.

Care must be taken to protect practitioner, assistant and patient. Pulpdent Porcelain Etch Gel has been buffered, gelled and packaged to minimize the dangers of hydrofluoric acid, provided it is used in accordance with these instructions.

For safe intra-oral use, carefully follow the Directions for Use.

WARNING: If Porcelain Etch Gel comes in contact with skin or soft tissue, rinse exposed area with copious amounts of water. Use high-speed evacuation system to remove water.

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