

# FILE-RITE™

## Canal Lubricant and Chelating Agent

- 4 x 5 gm syringes File-Rite
- 50 applicator tips
- Instructions for Use
- MSDS

**CAUTION:** Federal law restricts this device to sale by or on the order of a dentist.  
STORE IN COOL, DRY PLACE.

MADE IN USA

File-Rite is a 17% EDTA gel that has been buffered to a neutral pH. During instrumentation and debridement of the root canal, File-Rite is an effective calcium binding material for chelating and removing dentin from the canal walls, and it acts as a lubricant to minimize the binding and breaking of files.

### Directions for Use

1. Always place a rubber dam.
2. Remove cap from the syringe and place an applicator needle securely on the syringe.
3. Dispense File-Rite directly into the canal. For controlled dispensing, hold syringe barrel with fingertips, place plunger in palm, and gently pull back with fingers. File-Rite can also be placed directly on the file or into a dappen dish.
4. Instrument the canal. Reapply File-Rite and repeat as necessary until the canal is enlarged and shaped as desired.
5. Rinse canal thoroughly.
6. Discard needle and place cap back on the syringe. Use a new needle for each patient.



0459

### ***PULPDENT Corporation***

80 Oakland Street Watertown, MA 02471-0780 USA  
Tel: (800) 343-4342/(617) 926-6666/Fax: (617) 926-6262  
[www.pulpdent.com](http://www.pulpdent.com) [pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com)

XP-FILE-IN-03w