

# Pulpdent<sup>®</sup> Cavity Liner

The original premixed calcium hydroxide-methylcellulose preparation in liquid form.

---

## THREE-WAY PROTECTION

*Chemical – Physical – Biological*

---

### CHEMICAL PROTECTION

Stops chemical irritation to pulp by neutralizing acids from all filling materials. Bacteriostatic and bactericidal.

### PHYSICAL PROTECTION

Creates protective barrier between filling and dentin.

### BIOLOGICAL PROTECTION

Stimulates growth of sclerotic and reparative dentin.

### USES

Pulpdent Cavity Liner is placed on all exposed dentin, under fillings, in indirect pulp capping procedures and on dentin after crown preparations.

### DIRECTIONS

1. Shake bottle well making sure stainless steel mixing ball is moving.
2. Place a drop of Pulpdent Cavity Liner liquid onto slab and use explorer, spoon excavator or other instrument to place liner in cavity.
3. Dry with a slow stream of air. Pulpdent Cavity Liner appears chalk white when completely dry.
4. Wipe off excess from enamel margins with a damp pledget of cotton or with an explorer.



### ***PULPDENT Corporation***

80 Oakland Street Watertown, MA 02472 USA

Tel: (800) 343-4342/(617) 926-6666/Fax: (617) 926-6262  
www.pulpdent.com pulpdent@pulpdent.com

XP-PCL-IN-04w  
REV: 02/2012