

# The Spee-Dee Way to Build Up Broken Down Teeth

**PULPDENT Corporation**  
80 Oakland Street  
Watertown, MA 02472 USA  
(617) 926-6666  
(800) 343-4342  
(617) 926-6262 (fax)  
pulpdent@pulpdent.com  
[www.pulpdent.com](http://www.pulpdent.com)

PULPDENT CORPORATION  
Spee-Dee Build-Up

Multi-purpose products can simplify the already complicated dental armamentarium without compromising quality. In fact, procedures can be streamlined and quality can be improved. Rather than juggling materials and being concerned about the compatibility of various products, practitioners can be comfortable using a product with which they are familiar and that produces consistent results.


Spee-Dee Build-Up is an advanced dual cure core and tooth build up material that is self-adhesive, self-etching to dentin and moisture friendly. Its many uses include post cementation, direct bonded cores without a post, vital tooth build up for crown preps, base/liner under restorations, repair of lost fillings and broken splints, and patching up broken cusps and chipped porcelain.

Spee-Dee Build-Up is used as a one step post and core build up, ensuring a homogenous, one-piece internal structure that provides strength and monoblock continuity within the restoration.

Contrary to manufacturers' claims, traditional composite resins do not cut like dentin. Spee-Dee Build-Up is different. Practitioners consistently report that Spee-Dee Build-Up has the same resistance to the cutting bur as tooth structure. Rotary instruments transition smoothly from tooth to Spee-Dee. They cut accurately and do not ditch, gouge, gum up or chatter.

Spee-Dee Build-Up is packaged in 25mL and 50mL automix cartridges for convenient dispensing through automix tips.


## Post Cementation and Core Build-Up with Spee-Dee Build-Up

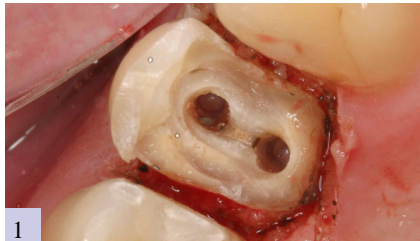


1 After endodontic treatment, a molar is prepared with two post holes.



2 Etch-Rite is applied to the prep and post holes for 15 seconds. Etching is optional.

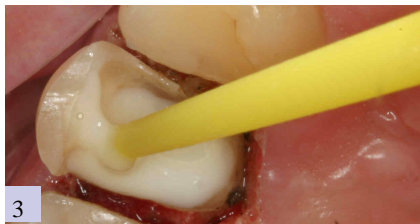


3 After rinsing, removing excess moisture and leaving the tooth slightly moist, Spee-Dee Build-Up is dispensed into the post holes and around the preparation.

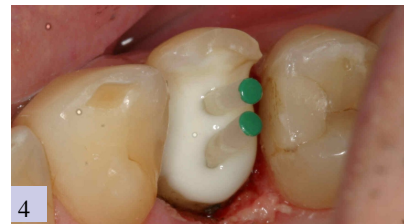

4 The posts are inserted with a twisting up and down motion to ensure uniform coverage of Spee-Dee Build-Up and then light cured for 20 seconds.



5 Spee-Dee Build-Up was applied around the posts to the occlusal level.

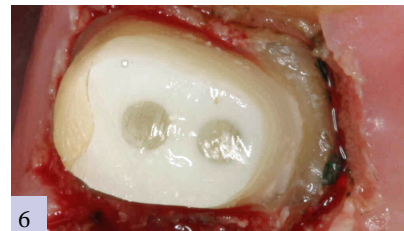

6 The post and core was trimmed and finished. Spee-Dee Build-Up really does cut like dentin.