PULPDENT®

Etch·All™

Dental Etching Gel

- 10% Phosphoric Acid
- Contains No Silica
- Thixotropic

CONTENTS:

- 4 1.2ml syringes etching gel
- 8 dispenser needles
- Instructions for use
- MSDS

Caution: Federal law restricts this device to sale by or on the order of a dentist.

For total etch of enamel and dentin

MADE IN USA

PULPDENT®

Etch·All™
Dental Etching Gel

PULPDENT®

Etch·All™

Dental Etching Gel • For Acid Etching of Enamel and Dentin

10% PHOSPHORIC ACID

Etch-All is a non silica, thixotropic phosphoric acid etch gel which stays in place and will not run onto exposed tissue. Dispensed from a syringe through pre-bent small gauge needles, it offers precise placement of gel even in relatively inaccessible areas. Etch-All washes off easily with water.

DIRECTIONS FOR USE

1. Cleanse the enamel surface to be etched with a slurry of pumice or non-oil containing prophylaxis paste. Rinse and dry thoroughly.
2. Attach dispensing needle to front of Etch-All syringe.
3. Apply Etch-All directly onto tooth surface to be etched. Leave in place for 30 seconds while agitating gently.
4. Rinse off Etch-All with copious amounts of water.
5. Leave surface moist or dry surface with clean, uncontaminated air, depending on technique.

WARNING:

If Etch-All contacts hands or soft tissue, rinse off immediately with copious amounts of water.

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PULPDENT Corporation