

# PREP-RITE™ RC

**Viscous EDTA Gel, 17%**

**Canal Lubricant and Chelating Agent**

- **4 x 5 gm syringes**
- **Instructions for Use**
- **MSDS**

CE 0459 Rx only



Prep-Rite RC is a viscous 17% EDTA gel that has been buffered to a neutral pH. It contains peroxide, which provides an effervescing action, and has a viscous consistency so that it can be easily picked up on a file. During instrumentation and debridement of the root canal, Prep-Rite RC is an effective calcium binding material for decalcifying the canal walls, and it acts as a lubricant to minimize the binding and breaking of files. Prep-Rite is water-based and easily rinses out of the canal.

## **Directions for Use**

1. Always place a rubber dam.
2. Remove cap from the syringe.
3. Dispense Prep-Rite RC onto a pad. For controlled dispensing, hold syringe barrel with fingertips, place plunger in palm, and gently pull back with fingers. Recap syringe immediately after use.
4. Dip the file into the Prep-Rite gel. Prep-Rite picks up on the file.
5. Instrument the canal and irrigate in the usual manner. Reapply Prep-Rite RC and repeat as necessary until the canal is enlarged and shaped as desired.
6. Rinse canal thoroughly.



**PULPDENT Corporation**

80 Oakland St.  
Watertown, MA 02472

[www.pulpdent.com](http://www.pulpdent.com) [pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com)

MADE IN USA  
Store in cool, dry place

XP-PRC-IN-02w  
REV: 01/2011