# PULPDENT® TempCanal™


A therapeutic treatment paste for routine canal treatment between office visits and whenever extended calcium hydroxide therapy is indicated. Pulpdent TempCanal has a pH > 12 and provides the proven bactericidal and bacteriostatic benefits of calcium hydroxide for successful root canal therapy. Use routinely as an antimicrobial intracanal dressing between office visits to disinfect the canal and help prevent flare-ups. Use TempCanal in complicated cases involving treatment of infection, periapical cysts, lesions and abscesses, internal and external root resorption, traumatic injury, apexification and other instances where extended calcium hydroxide therapy is indicated.

## INSTRUCTIONS FOR USE

1. Prepare and cleanse canal in conventional manner.
2. Select dispenser needle that corresponds to size of last file used. Remove white cap to expose needle hub.
3. Self thread needle fully onto syringe. Remove colored sheath from needle.
4. To start flow, turn screw plunger slowly allowing time for material to flow through needle. Turn plunger counter-clockwise to stop flow.
5. To avoid overfill, insert needle directly into canal at least 2mm short of the apex. Needle can be bent for easy access. Do not crimp needle.
6. Fill canal completely by turning plunger while slowly withdrawing needle.
7. Seal access opening with your preferred cement.
8. TempCanal is easily removed with file and irrigation.

NOTE: To extend shelf life, place a few drops of water in the cap and recap syringe promptly after each use.

## CLINICAL SITUATIONS WHERE CALCIUM HYDROXIDE CAN BE USED

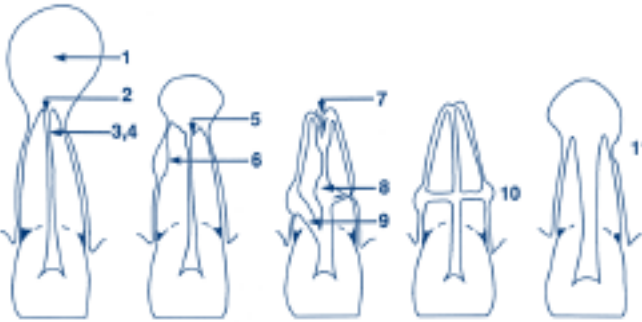- |  |  |
|--|--|
| <ol style="list-style-type: none"> <li>1. exudation control</li> <li>2. large periapical lesions</li> <li>3. antibacterial dressing</li> <li>4. temporary root filling</li> <li>5. apical inflammatory resorption</li> <li>6. inflammatory resorption following trauma</li> <li>7. apical internal resorption</li> </ol> | <ol style="list-style-type: none"> <li>8. internal-external root resorption</li> <li>9. perforations</li> <li>10. transverse root fractures</li> <li>11. incompletely developed pulp less teeth<br/><i>(Heithersay, G.S. Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975)</i></li> </ol> |
|--|--|

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**PULPDENT Corporation**
80 Oakland Street • Watertown, MA 02471-0780 • U.S.A.