

**PULPDENT®**

**WINK™**

Water soluble lubricant and release agent for professional dental use

---

**PRODUCT DESCRIPTION**

WINK is a highly effective, greaseless, water soluble lubricant and release agent for all dental applications. Applied intraorally or to the face and lips with a cotton pellet or other suitable applicator, WINK goes on smoothly and evenly, is easily thinned with compressed air, and washes off quickly with water. It replaces petroleum jelly and other greasy, lumpy and hard to remove materials.

WINK helps prevent the distortion of impressions caused by torque and stresses during removal of impressions from the mouth.

**INDICATIONS**

- Release agent for all impression materials
- Release agent for final impressions for removable partial dentures
- Release agent for cement and impression material from face and tissue
- Lubricant for outside surface of restorations for fast removal of set cement
- Lubricant for build ups, teeth and preps when making temporary restorations
- Lubricant for rubber dam to facilitate placement without tearing
- Lubricant for tight fitting implant abutments during try-in phase
- Release agent for buccal surfaces of dentures during relining procedures
- Lubricant to prevent drying of face and lips during long procedures

**INSTRUCTIONS FOR USE**

1. Remove black shipping plug from spout prior to first use
2. Dispense a small amount of WINK into a dappen dish or mixing well.
3. Apply a thin film of WINK on the teeth, soft tissue or restorative surface. Air thin if desired.
4. WINK easily washes off with water.

**CAUTION**


- Avoid getting WINK under the margin of a restoration during cementation.
- Avoid getting WINK on the tooth prep when a final impression is being made for a permanent restoration.

**NOTE**

- Apply WINK on the tooth prep when making a temporary restoration.

**ALLERGY WARNING**

Wink contains soy-based component.

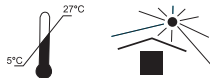

**PULPDENT Corporation**

80 Oakland Street • Watertown, MA 02472 • USA

Tel: (800) 343-4342 / (617) 926-6666 / Fax: (617) 926-6262

www.pulpdent.com • pulpdent@pulpdent.com

XP-WINK-IN-03w  
REV. 08/2011