Porcelain Prep Kit

CONTENTS:

- 1 1.2mL syringe Porcelain Etch Gel
- 1 1.2 mL syringe Kool-Dam™
- 1 1.2 mL syringe Dry-Rite™
- 1 1.2mL syringe Silane Bond Enhancer
- Applicator tips
- Instructions for use
- MSDS

Caution: Federal law restricts this device to sale by or on the order of a dentist.

Store in a cool, dry place

MADE IN USA

To prepare porcelain surfaces (teeth, inlays, veneers) for bonding

Porcelain Prep Kit

The PULPDENT PORCELAIN PREP KIT contains the necessary ingredients for preparing porcelain veneers and inlays and fractured porcelain teeth for bonding.

CAUTION:

Porcelain Etch Gel contains 9.6% hydro-fluoric acid, a powerful and aggressive chemical that produces irritating fumes and must be handled with extreme care. Ingestion, contact with skin and eyes, and chronic inhalation of fumes are dangerous. Care must be taken to protect practitioner, assistant and patient. Pulpdent Porcelain Etch Gel has been buffered, gelled and packaged to minimize the dangers of hydrofluoric acid, provided it is used in accordance with these instructions. For safe intra-oral use, carefully follow the Directions for Use.

WARNING:

If Porcelain Etch Gel comes in contact with skin or soft tissue, rinse exposed area with copious amounts of water for at least two minutes. Use high-speed evacuation system to remove water.

INSTRUCTIONS FOR USE

1. Protective eyewear must be worn by practitioner, assistant and patient. Protective gloves must be worn by practitioner and assistant.
2. Place rubber dam to isolate operative site and protect soft tissue.
3. Apply a coat of Kool-Dam 1-2 mm thick on the gingiva or tooth surfaces requiring protection, if needed.
4. Light cure for 10 seconds. Cures with all lights.
5. Assistant must stand ready at chairside and hold vacuum tip 2-3 inches from porcelain surface during the etching process to evacuate acid fumes.
6. Remove syringe cap from Porcelain Etch Gel and place a pre-bent dispenser needle on the syringe. Carefully start the flow of Porcelain Etch Gel on a pad and immediately dispose of this sample. Do not start flow in patient's mouth. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between thumb and forefinger while resting plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Never force plunger. Use a new dispenser needle for each patient.
7. Carefully apply Porcelain Etch Gel to porcelain surface and leave for one minute while evacuating fumes with vacuum. Cap syringe immediately after use.
8. Remove etch gel with vacuum. Rinse treated surface with copious amounts of water and dry with clean, uncontaminated, oil-free and moisture-free compressed air.
9. Using red dropper tip, apply Pulpdent Dry-Rite drying agent to treated surface. To promote rapid drying, use a gentle flow of clean, oil-free air if desired. Cap Dry-Rite immediately after use.
10. Apply Pulpdent Silane Bond Enhancer with red dropper tip and allow to dry. Apply a gentle flow of clean, oil-free and moisture-free air, if desired. Cap syringe immediately after use.
11. Proceed with repair or cementation of porcelain teeth, inlays or veneers using your preferred unfilled resin bonding agent and restorative resin or resin cement.

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PULPDENT Corporation