This 17% solution of EDTA has been buffered to a neutral pH. It is an effective chelating agent to assist in the instrumentation of root canals and in smear layer removal.

For Root Canal Cleaning And Shaping

EDTA solution helps decalcify the dentin in the canal wall. This makes it easier to remove dentin with files and reamers and helps the dentist enlarge and shape the canal for final filling with sealer and gutta percha.

Instructions for use:

1. Apply EDTA at entrance of canal with pellet or pipette, and file and ream canal in conventional manner.
2. Reapply EDTA as necessary until canal is enlarged to the desired size.
3. Rinse canal thoroughly so that all EDTA is removed

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from the canal.

For Smear Layer Removal Prior to Placement of Endodontic Posts

After drilling a root canal for a post, the dentin walls of the post preparation are covered with a smear layer of dental debris. This smear layer can be removed by rinsing the cavity with Pulpdent EDTA Solution, 17%.

Instructions for use:

1. Prepare canal to receive a post.
2. Using a pipette, rinse canal with EDTA and immediately evacuate the EDTA with high suction for 10 seconds.
3. Rinse canal with water and remove excess water with air and/or paper points.

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Material Safety Data Sheet -

Identity

Trade Name: EDTA Solution, 17%
Chemical Description: 17 % Ethylenediaminetetraacetic acid in aqueous solution
Product Use: Dental chelating agent used to debride root canals

Section I

Pulpdent Corporation 80 Oakland Street	Phone Numbers 24 Hour Emergency: 1-800-535-5053
Watertown, MA 02472 USA	Customer Service Numbers: 1-800-343-4342/1-617-926-6666
Date Prepared: August 1, 2011	

Section II

Ingredients Ethylenediaminetetraacetic acid	CAS RN 60-00-4
---	--------------------------

Dot Hazard Classification: Not regulated
Whmis Classification: None
Nfpa Hmis Rating: Health: 0 Flammability: 0 Reactivity: 0

Section III - Physical / Chemical Characteristics

Odor: None	Specific gravity: 1.00
pH: 7 ± 0.2	
Appearance: Clear liquid.	Solubility in water: Soluble
Odor threshold: Not available	

Section IV - Fire and Explosion Hazard

Flash point: Not applicable
Flammable limits: Not applicable
Autoignition temperature: Not applicable
Hazardous Combustion Products: Ammonia, nitrogen oxides under fire conditions.
Extinguishing Media: Water fog, alcohol foam, CO₂, dry chemical
Special Fire Fighting Procedures: Positive pressure, self-contained

breathing apparatus.
Unusual Fire and Exposure Hazards: None

Section V - Reactivity Data

Stability: Stable **Conditions to avoid:** None
Incompatibility: Strong oxidizing agents
Hazardous Decomposition Products: Ammonia, nitrogen oxides under fire conditions.
Hazardous Polymerization: Will not occur **Conditions to avoid:** None

Section VI - Health Hazard Data

Summary of Acute Hazards: Minimal health hazard under normal conditions of use.

Route of Exposure Inhalation Eye contact Skin absorption Skin contact Ingestion:	Signs and Symptoms None May cause transient irritation. None Non-irritating None expected under normal conditions of use.
--	---

Summary of Chronic Hazards: None known
Carcinogenicity: Not carcinogenic in long term animal studies.
Teratogenicity: None known.
Mutagenicity: None known.
Reproductive toxicity: None known.

Emergency First Aid Procedures
Inhalation: Not applicable
Eye contact: Irrigate immediately with water. Seek medical care for persistent irritation.
Ingestion: For large amounts: Consult physician.
Skin contact: Wash off with running water.

Section VII - Precautions for Safe Handling and Use

Handling and Storage Precautions: None needed.
Release or spill: Wear gloves and safety glasses. Pick up with absorbent material, such as paper or cloth towels.
Waste Disposal Method: Follow all government regulations.
Other Precautions: Wash hands after use.

Section VIII - Control Measures

Respiratory Protection: Not necessary under normal conditions of use.

Ventilation: No special ventilation required under normal conditions of use.

Protective Gloves: Recommended

Eye Protection: Safety glasses are recommended under normal conditions of use.

Other Protective Clothing or Equipment: Emergency eye wash fountain should be close by and maintained.

Work/Hygienic Practices: Wash hands after use.

The information presented herein is believed to be factual as it has been derived from the works of persons believed to be qualified experts. However, nothing contained in this information is to be taken as a warranty or representation for which Pulpdent Corporation bears legal responsibility. The user should review any recommendations in the specific context of the intended use to determine whether they are appropriate.

R_x only

CE
0459

PULPDENT[®]

www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.
(617) 926-6666 or (800) 343-4342 • Fax (617) 926-6262

XP-EDTA-IN-14w
REV: 08/2011