Forendo™ Paste

Calcium hydroxide with Iodoform

Bactericidal - Bacteriostatic - Radiopaque - pH>12

PRODUCT DESCRIPTION

Forendo Paste is a temporary endodontic filling material that promotes healing and inhibits bacterial growth in the root canal. It is recommended for routine use as a disinfecting root canal dressing between office visits to prevent flare-ups and for treatment of infected root canals, abscesses, periapical lesions, perforations, exudation, root resorption, traumatic injuries, and for apexification and apexogenesis.

Forendo Paste can be used alone as a temporary canal disinfectant, for vital and non-vital pulpotomies, and for pulpectomies in deciduous teeth.

Forendo Paste is a non-setting, premixed paste that contains calcium hydroxide and iodoform in a silicone oil base. It has a pH>12 and is radiopaque.

INSTRUCTIONS FOR USE

Temporary Canal Treatment

1. Using good endodontic procedures, cleanse and prepare the root canal in the conventional manner. Dry the canal.
2. Place a new disposable applicator tip securely on the luer lock syringe.
3. Fill the canal by applying gentle pressure on the syringe plunger. Do not force the plunger. The paste can also be placed into the canal with a lentulo spiral.
4. Remove excess at the orifice with a sterile cotton pellet.
5. Seal access opening with your preferred cement.
6. Discard applicator tip and cap syringe immediately after use.
7. Forendo Paste is easily removed with file and irrigation with sodium hypochlorite.

Treatment of Primary Teeth

1. Follow instructions above for Temporary Canal Treatment.
2. It is not necessary to remove Forendo Paste from the root canal. The material will resorb with the roots in deciduous teeth.

STORAGE AND HANDLING

- When dispensing from the syringe, do not force the plunger. Apply light pressure only. If material does not dispense, remove needle, purge some material from the syringe, and place a new needle on the syringe.
- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap syringe immediately after use.

CAUTION

For professional use only by dentists. Wear protective eyewear and surgical gloves. Avoid contact with eyes, mucous membranes and skin. If contact occurs, flush with copious amount of water. If contact occurs with eyes, flush with copious amounts of water and seek medical attention immediately.

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PULPDENT Corporation