# Pulpdent® Paste

## The Original Pre-Mixed Calcium Hydroxide Dressing

Pulpdent Paste, with pH > 12, is bacteriostatic, bactericidal and radiopaque. It is applied directly to the exposed pulp in all pulp capping, pulpotomy and pulpal curettage procedures. Pulpdent Paste stimulates the pulp to form a new dentin bridge which is visible clinically after 2-3 months.

Pulpdent Paste is used routinely in root canal therapy as an antibacterial intracanal dressing between office visits and to treat periapical lesions, abscesses, root resorption, traumatic injuries and other complicated cases.

### INSTRUCTIONS FOR USE

#### For Vital Pulp Therapy


1. Remove plastic cap. Securely place blunt needle on syringe. You may bend needle to facilitate access to cavity. Alternatively, dispense paste on a pad and place it into the cavity with an appropriate instrument.
2. For pulpotomy, place 2–3mm of Pulpdent Paste onto pulp. For pulp capping, place 1–2mm of Pulpdent Paste at exposure site.
3. Dry with stream of air to create a "crust".
4. Gently tamp Pulpdent Paste with a pledget of cotton to assure contact with the pulp. Again dry with stream of air.
5. Flow thin mix of hard setting cement over the Pulpdent Paste. Place final restoration.

#### For Root Repair

1. Fill root canal with Pulpdent Paste using the needles provided, the Pulpdent Pressure Syringe, a lentulo spiral or blunt paper point.
2. Cover with a pledget of cotton and a temporary filling material.
3. Pulpdent Paste can be removed by minimal filing and irrigation and replaced as necessary until the desired results have been achieved. The root canal can then be obturated by an appropriate method.

**NOTE:** To extend shelf life, place a few drops of water in the cap and recap syringe promptly after each use.

### CLINICAL SITUATIONS WHERE CALCIUM HYDROXIDE CAN BE USED

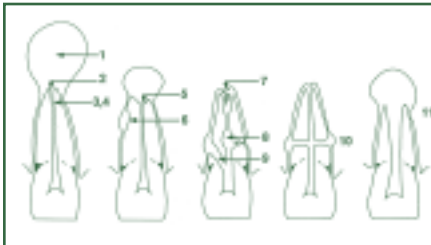1. exudation control
  2. large periapical lesions
  3. antibacterial dressing
  4. temporary root filling
  5. apical inflammatory resorption
  6. inflammatory resorption following trauma
  7. apical internal resorption
  8. internal-external root resorption
  9. perforations
  10. transverse root fractures
  11. incompletely developed pulpless teeth
- (Heithersay, G.S. Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975)

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## ***PULPDENT Corporation***