This 17% solution of EDTA has been buffered to a neutral pH. It is an effective chelating agent to assist in the instrumentation of root canals and in smear layer removal.

#### FOR ROOT CANAL CLEANING AND SHAPING

EDTA solution helps decalcify the dentin in the canal wall. This makes it easier to remove dentin with files and reamers and helps the dentist enlarge and shape the canal for final filling with sealer and gutta percha.

#### Instructions for use:

1. Apply EDTA at entrance of canal with pellet or pipette, and file and ream canal in conventional manner.
2. Reapply EDTA as necessary until canal is enlarged to the desired size.
3. Rinse canal thoroughly so that all EDTA is removed from the canal.

#### FOR SMEAR LAYER REMOVAL PRIOR TO PLACEMENT OF ENDODONTIC POSTS

After drilling a root canal for a post, the dentin walls of the post preparation are

covered with a smear layer of dentinal debris. This smear layer can be removed by rinsing the cavity with Pulpdent EDTA Solution, 17%.

#### Instructions for use:

1. Prepare canal to receive a post.
2. Using a pipette, rinse canal with EDTA and immediately evacuate the EDTA with high suction for 10 seconds.
3. Rinse canal with water and remove excess water with air and/or paper points.

**Note:** Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

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