

Pulpdent[®] Cavity Liner

The original premixed calcium hydroxide-methylcellulose preparation in liquid form.

THREE-WAY PROTECTION

Chemical – Physical – Biological

CHEMICAL PROTECTION

Stops chemical irritation to pulp by neutralizing acids from all filling materials.

PHYSICAL PROTECTION

Creates protective barrier between filling and dentin.

BIOLOGICAL PROTECTION

Stimulates growth of sclerotic and reparative dentin.

USES

Pulpdent Cavity Liner is placed on all exposed dentin, under fillings, in indirect pulp capping procedures and on dentin after crown preparations.

DIRECTIONS

1. Shake bottle well making sure stainless steel mixing ball is moving.
2. Place a drop of Pulpdent Cavity Liner liquid onto slab and use explorer, spoon excavator or other instrument to place liner in cavity.
3. Dry with a slow stream of air. Pulpdent Cavity Liner appears chalk white when completely dry.
4. Wipe off excess from enamel margins with a damp pledget of cotton or with an explorer.



PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.