5% Sodium Fluoride with CXP™ • CXP™ = Xylitol-coated Calcium and Phosphate for unmatched fluoride release

EMBRACE™ Varnish

The complete varnish with bioavailable calcium, phosphate and fluoride

PRODUCT DESCRIPTION AND INTENDED USE

EMBRACE™ Varnish is a resin-based varnish that provides sustained, time-release of fluoride, calcium and phosphate ions over a period of 4 hours or longer. The calcium and phosphate salts are nano-coated with xylitol and are bioavailable. Exposure to saliva dissolves the xylitol and releases the calcium and phosphate ions, which continuously react with fluoride ions the saliva to form protective fluorapatite on the teeth.

EMBRACE™ Varnish contains 5% sodium fluoride, which is equivalent to 22,600 ppm fluoride (22.6 mg fluoride/mL). The material is tooth shade. It dries quickly forming a smooth thin film on the teeth.

INDICATIONS

EMBRACE™ Varnish treats hypersensitivity by releasing fluoride, calcium and phosphate ions that precipitate on and occlude dental tubules and fill superficial, non-carious enamel lesions.

CONTRAINDICATIONS

Ulcerative gingivitis and stomatitis

NOTE:

Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/Dental-Products/ucm404472.htm>

PRECAUTIONS

Limit ingestion by applying only as much varnish as necessary to cover tooth surfaces with a thin film. Adult patients should refrain from taking prescription fluoride supplements for a period of 24 hours following treatment, and children should refrain for 2-3 days.

INSTRUCTIONS FOR USE

1. Use back of cap to pierce seal on the tube.
2. Dry the teeth with a gauze pad. It is best if teeth are at least minimally clean, but it is not necessary to do a prophylaxis.
3. On a pad or mixing well, dispense 0.25 mL varnish for children and 0.4mL for adults.
4. Using a brush, apply a very thin coat of varnish on the desired tooth surfaces. It is not necessary to use all the varnish.
5. It is best if patients refrain from eating hard foods or drinking hot liquids or alcohol for 3-4 hours after treatment. Eating soft foods for the rest of the day is advantageous. Instruct patients not to brush their teeth for at least four hours.
6. EMBRACE Varnish can be removed by brushing and flossing.

WARNING

- Federal law restricts this device to sale by or on the order of a dentist or physician.
- EMBRACE Varnish may be irritating to respiratory tract if inhaled or to eyes or skin on contact. May be a sensitizer with prolonged and/or repeated contact. See MSDS for further information.
- Dental practitioners should wear protective eyewear and gloves. Wash hands after use. Emergency eye wash fountain should be available in the office.
- EMBRACE Varnish contains no soy or gluten products. It contains a processed pine tree extract and not an extract of pine nut. Patients with allergies should consult their physician.

STORAGE

Store original packages at cool room temperature 5-25°C / 41-77°F. Avoid extremes of temperature.

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PULPDENT Corporation
80 Oakland Street • Watertown, MA 02472 • U.S.A.