


PULPDENT®

Sparkle™

Diamond Polishing Paste

 4 x 1.2mL Sparkle Diamond Polish

 **PULPDENT Corporation**
80 Oakland Street • Watertown, MA 02472 • U.S.A
www.pulpdent.com • pulpdent@pulpdent.com

R_x only



CE



PULPDENT®
Sparkle™
Diamond Polishing Paste

PULPDENT®

Sparkle™ Diamond Polishing Paste

For polishing porcelain, gold, metal and composite to a high luster finish

Sparkle is specially formulated to produce unsurpassed results on porcelain, gold, metal and composite. It does not splatter, and it washes off easily.

Sparkle is packaged in syringes for easy dispensing. It is available in kits of four 1.2ml syringes, and it is also available in a 3ml syringe.

DIRECTIONS FOR USE

1. Apply to surface or restoration with brush, rubber cup or felt wheel.
2. Polish to high luster finish.
3. Wash off with water.
4. Keep syringe capped when not in use.

PULPDENT Corporation • 80 Oakland Street • Watertown, MA 02472 • U.S.A.
www.pulpdent.com • pulpdent@pulpdent.com

XK-SPARK-IN-08w
REV: 05/2022

SAFETY DATA SHEET



www.pulpdent.com/info