
Porcelain Etch Gel

9.6% HF

PULPDENT®
80 Oakland Street • Watertown, MA 02472 • U.S.A.

Rx only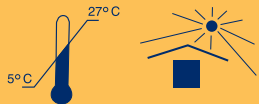

Porcelain Etch Gel 9.6% Hydrofluoric Acid

Does not stain resins or resin cements.

Pulpdent Porcelain Etch Gel is a 9.6% hydrofluoric acid gel for etching porcelain veneers and inlays or for etching fractured porcelain surfaces prior to cementation or bonding with resins and resin cements. It can also be used to etch hybrid composites.

PRODUCT DESCRIPTION
9.6% hydrofluoric acid gel

INDICATIONS

For etching porcelain surfaces and hybrid composites prior to cementation or bonding with resins and resin cements.

CONTRAINDICATIONS

Do not place in contact with teeth and soft tissues

DIRECTIONS FOR USE

1. Protective eyewear must be worn by practitioner, assistant and patient. Protective gloves must be worn by practitioner and assistant.
2. Place rubber dam to isolate operative site and protect soft tissue.
3. Assistant must stand ready at chairside and hold vacuum tip 2-3 inches from porcelain surface during the etching process to evacuate acid fumes.
4. Securely attach 25-gauge luer lock applicator tip to syringe. Carefully start the flow of Porcelain Etch Gel on a pad and immediately dispose of this sample. Do not start flow in patient's mouth. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between thumb and forefinger while resting plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Use a new dispenser needle for each patient.
5. Using gentle pressure, carefully apply Porcelain Etch Gel to porcelain surface and leave for one minute while evacuating fumes with vacuum. Some ceramic materials are resistant to etching. To achieve a surface suitable for bonding to these materials, air abrasion is recommended. Creating mechanical retention is advantageous. Cap syringe immediately after use.
6. Remove gel with vacuum tip and rinse treated surface with copious amounts of water. Dry porcelain surface with clean uncontaminated air.
7. If using Embrace™ First-Coat™, follow instructions for use for First-Coat or the

Embrace™ WetBond™ Restoration and PFM Repair Kit. If using Pulpdent Silane Bond Enhancer, follow instructions for use for silane.

8. Proceed with cementation or bonding with resins and resin cements.

CAUTION

Porcelain Etch Gel contains 9.6% hydrofluoric acid, a powerful and aggressive chemical that produces irritating fumes and must be handled with extreme care. Ingestion, contact with skin and eyes, and chronic inhalation of fumes are dangerous. Care must be taken to protect practitioner, assistant and patient. Pulpdent Porcelain Etch Gel has been buffered, gelled and packaged to minimize the dangers of hydrofluoric acid, provided it is used in accordance with these instructions.

WARNING

If Porcelain Etch Gel comes in contact with skin or soft tissue, rinse exposed area with copious amounts of water. Use high-speed evacuation system to remove water.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB
My Trade Schweiz GmbH