# Silane

## Bond Enhancer

Rx only

 4 x 1.2ml Silane

 PULPDENT®
80 Oakland Street • Watertown, MA 02472 • U.S.A.  
www.pulpdent.com • pulpdent@pulpdent.comXK-SIL-IN-26w  
REV: 10/2022

PULPDENT®

## Silane Bond Enhancer

### PRODUCT DESCRIPTION

PULPDENT SILANE is a single-component material which is used to promote increased bond strength between organic resins and porcelain. To achieve desired results, the etched porcelain surface must be completely dry and free of residual moisture before applying silane.

### INDICATIONS

For use on etched and dried porcelain surfaces prior to placement of organic resins

### CONTRAINDICATIONS

Do not place on moist porcelain surfaces

### DIRECTIONS FOR USE

- Carefully follow instructions for Pulpdent Porcelain Etch Gel, and etch porcelain surface to be treated. Rinse and dry.
- Apply Pulpdent Dry-Rite™ drying agent to etched surface to assure that it is completely dry. Residual moisture will adversely affect bonding.
- Place 22-gauge red dropper tip on silane syringe and apply Pulpdent Silane Bond Enhancer to treated porcelain surface. Allow to dry. A gentle flow of uncontaminated, oil-free and moisture-free air may be applied to accelerate drying. Cap syringe immediately after use. To check air line for contamination, blow air onto the surface of dental mirror. If streaks of water or droplets of oil appear, do not use the air line for this procedure.
- Maintain a clean, dry field. Apply unfilled resin bonding agent and proceed with your preferred restorative composite or resin cement.

**Note:** Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient.

For additional information, refer to: <http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

### STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

### REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation  
80 Oakland Street, Watertown, MA 02472 USA  
Tel: 617-926-6666 / Fax: 617-926-6262  
E-Mail: [pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com)

**Note:** Refer to the Safety Data Sheet for a list of materials present in this product.

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**Hofmeister Dental B.V.**
**Lifco Dental AB**
**UNIDENT AB**