Multi-Cal™

Calcium Hydroxide Preparation

PULPDENT® 80 Oakland Street • Watertown, MA 02472 • U.S.A.
www.pulpdent.com • pulpdent@pulpdent.com

XP-MULTI-IN-16w
REV: 10/2022

PULPDENT®

MULTI-CAL™

Non-setting • pH > 12 • Radiopaque

PRODUCT DESCRIPTION

Pulpdent Multi-Cal is a smooth, creamy, non-setting calcium hydroxide preparation in an aqueous hydroxyethylcellulose base with barium sulfate. It is recommended for all clinical applications where calcium hydroxide paste is indicated. It is a dressing, not a hard base, and it provides all the same clinical results as Pulpdent Paste and TempCanal Enhanced.

INDICATIONS

Direct Pulp Capping, Pulpal Curettage, Pulpotomy, Dentin Bridge Formation, Temporary and Intermediate Root Canal Treatment, Periapical Cysts, Abscesses and Lesions, Root Resorption, Root Perforations, and Apexification

CONTRAINDICATIONS

Not for use as a hard base

VITAL PULP TREATMENT

Multi-Cal is applied directly on the pulp in all pulp capping, pulpotomy and pulpal curettage procedures. It stimulates the pulp to form a new dentin bridge which is visible clinically after 2 - 3 months.

DIRECTIONS FOR DIRECT PULP CAPPING:

- Place a 22-gauge applicator tip on the Multi-Cal syringe and apply a very thin film of Multi-Cal on all exposed dentin and dry with a slow flow of air.
- Place a bead of Multi-Cal 3mm thick directly over the exposed pulp and dry with a slow flow of air until a crust appears. Multi-Cal does not set hard.
- Gently tamp the paste with a pellet of cotton to make sure it is in contact with the pulp.
- Place a thin mix of hard setting cement over Multi-Cal.
- After cement hardens, place final restoration.

Note: For vital pulpotomy and pulpal curettage procedures, pack pulp gently with sterile cotton until blood clots, then proceed as above. A local anesthetic may be used.

ROOT CANAL TREATMENT

Use routinely as an intracanal dressing between office visits and as a dressing for cysts, abscesses, lesions, etc. Also use for apexification and other hard tissue formation.

DIRECTIONS FOR ROOT CANAL THERAPY:

- Prepare and cleanse canal in conventional manner.
- Place a 22-gauge applicator tip on the Multi-Cal syringe and fill canal with Multi-Cal.
- Seal access opening with your preferred cement.
- Multi-Cal is easily removed with file and irrigation.
- In cases requiring extended root canal therapy, change dressing every month.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

Note: Use a new applicator tip for each patient. To extend shelf life of Multi-Cal, place a drop of water in the cap and recap syringe promptly after each use.

CAUTION

May be irritating to eyes, skin, respiratory system, gastrointestinal tract or mucous membranes, especially in large quantities and/or with prolonged exposure. For use by dental professionals only. Wear gloves, safety glasses; wash hands after use. Avoid unnecessary exposure.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA


Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

CLINICAL SITUATIONS WHERE CALCIUM HYDROXIDE CAN BE USED

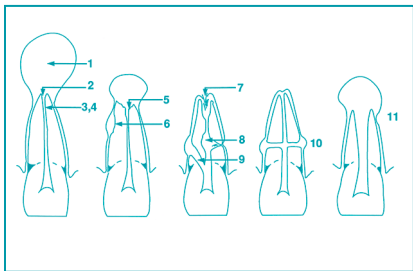- exudation control
- large periapical lesions
- antibacterial dressing
- temporary root filling
- apical inflammatory resorption
- inflammatory resorption following trauma
- apical internal resorption
- internal-external root resorption
- perforations
- transverse root fractures
- incompletely developed pulp less teeth

(Heithersay, G.S. Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975)

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