SNOOP™

THE CARIES DETECTIVE

Rx only

Caries Detecting Dye

Snoop is a propylene glycol based caries detector used to assist the dentist in identifying and removing infected carious dentin. Its dark blue color contrasts with dentin and is easily distinguished from the pulp. Researchers have reported that carious dentin is made up of two distinct layers. There is an outer layer of infected carious dentin which is soft, discolored, nonvital, nonsensitive and cannot remineralize. This outer layer is stained by Snoop and should be removed. There is also an inner layer of uninfected carious dentin (also referred to as affected dentin) which is vital and sensitive, not infused with bacteria, and may be somewhat softened and demineralized, but which is capable of remineralizing. This inner layer is not stained by Snoop and should not be removed. Research indicates that the dye in the caries detector bonds to the denatured collagen which is present in the outer infected dentin but which is not present in inner uninfected dentin and normal dentin. Only the outer infected dentin is stained. As a result, Snoop makes it easy to distinguish between outer and inner dentin and enables the dentist to perform a conservative cavity preparation removing only the outer infected dentin.

PRODUCT DESCRIPTION

Dark blue caries detecting dye in propylene glycol liquid formulation

INDICATIONS

For placement of carious dentin to distinguish outer infected dentin from inner affected dentin

CONTRAINDICATIONS

Do not place on dentin for longer than 10 seconds

DIRECTIONS FOR USE

- Use of rubber dam is recommended.
- Use Snoop as early in the cavity preparation procedure as possible to avoid removing too much tooth structure.
- Rinse treatment area with water and dry.
- Place applicator straw firmly into dropper spout. Using gentle pressure, carefully apply SNOOP directly to the cavity for 10 seconds. Alternatively, hold bottle vertically and dispense a drop of SNOOP into a mixing well. Using a small sponge, brush or paper point, carefully apply SNOOP to the cavity for 10 seconds.
- After 10 seconds, immediately apply suction and rinse the cavity with water, then dry.
- Remove the stained dentin with an excavator or a low speed handpiece with a round bur.
- Repeat procedure until no staining occurs.

WARNING

• Snoop contains dye that stains. Handle with care. • Apply Snoop only to the tooth to be stained and not to adjacent teeth. • If Snoop comes in contact with clothes, carpets or furniture, wash immediately with soap and water. • If Snoop comes in contact with eyes, flush with copious amounts of water for 15 minutes and seek medical attention. • Wear protective eye wear. Use rubber dam. Drape patient with plastic bib • Keep out of reach of children.

STORAGE AND HANDLING

• Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition. • Shelf life of unopened product: 2 years from date of manufacture. • Re-cap immediately after use.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which

XP-SNOOP-IN-22w
REV:11/2022

the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Ravelli S.p.A.
Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB